




Flexipet[®]

MICRO-MANIPULATION PIPETTE

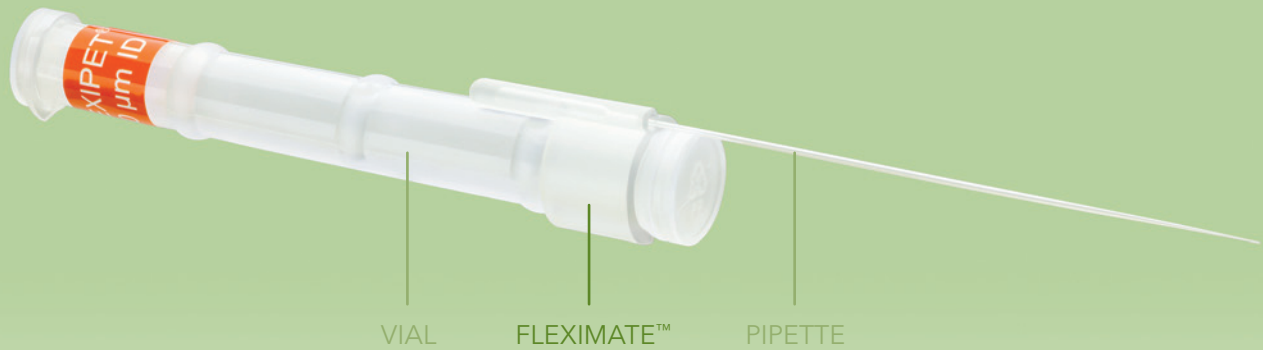

Manipulate oocytes and embryos with ease.

Partner the Flexipet[®] with the Microcise™ Fleximate™—an ergonomic, user-friendly accessory for denuding and manipulating oocytes and embryos more easily.



Flexipet[®]

MICRO-MANIPULATION PIPETTE



Color-coded containers make finding the right size easier.

Distinct labels on each Flexipet container correspond to pipettes that range in diameter from 80 µm to 600 µm, so finding the right size is simple.

Made from flexible polycarbonate that resists scratching surfaces and breaking, the pipettes are manufactured with the highest level of consistency and tested for single-cell MEA survivability and LAL toxicity.



Microcise[™] Fleximate[™]


FLEXIPET[®] ASPIRATION TOOL



Denude and manipulate more comfortably and efficiently.

Designed to create a vacuum for a Flexipet, the disposable Microcise Fleximate enables aspiration of an oocyte or embryo and denuding of the oocyte-cumulus complex.

Ergonomic and practical, the Fleximate accepts any size Flexipet pipette and sits flush on the Flexipet container or finger. Made from 100% silicone, it is comfortable and easy to use—which makes it helpful for manipulating oocytes and embryos more efficiently.

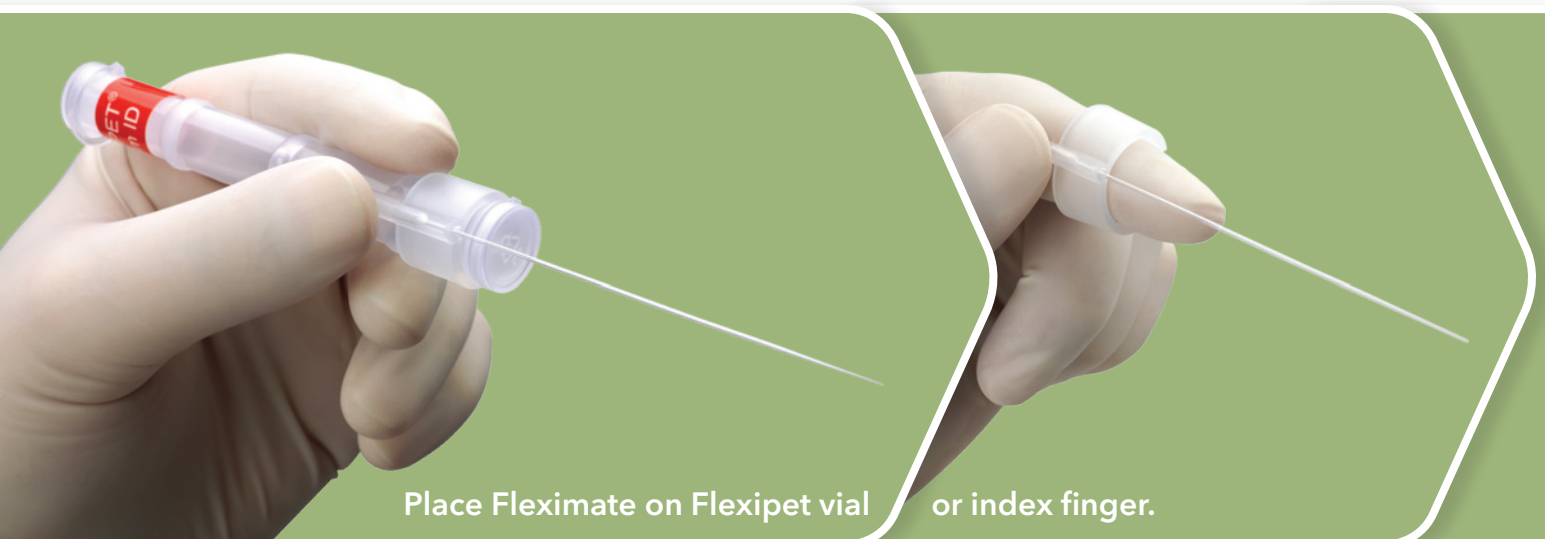

Place Fleximate on Flexipet vial

or index finger.

Features and Benefits

- Container allows convenient release of one pipette at a time.
- Pipettes available in various diameters for effective denuding or manipulation of embryos.
- Color-coded containers make identification of correct size pipette easier.

Features and Benefits

- Designed to accept any size Flexipet.
- Made to fit flush over the Flexipet holder or index finger.
- Facilitates ergonomic denuding or manipulation of an oocyte or embryo.
- Provides great tactile feedback and control.
- Quick and easy to use.
- Single-use product helps prevent cross-contamination.

Cook Medical—Simplifying procedures for patients and clinicians.

TO USE THE FLEXIMATE

- 1 Pipette at least 50 μ L of medium on the bottom of the petri dish.
- 2 Place Fleximate on large end of Flexipet holder or on the index finger with open end toward dish.
- 3 Insert Flexipet into Fleximate.
- 4 Partially compress bulb, then submerge Flexipet tip in media/wash buffer.
- 5 Slowly release bulb to begin medium aspiration. It is recommended that medium fill only half to three-quarters of Flexipet once bulb is fully released.
- 6 With medium now filling half to three-quarters of the Flexipet, begin oocyte/embryo aspiration or OCC denuding.
- 7 Partially compress the bulb and release some medium into the micro drop.
- 8 Slowly release bulb to aspirate cell. To avoid air bubbles, do not completely expel fluid from the Flexipet.
- 9 Repeat step 7 as needed.



Fertility specialists around the world can rely on Cook Medical's expertly crafted assisted reproduction products. Each one is engineered through collaborative research to meet the evolving and exacting needs of this delicate process and optimize outcomes for patients and their families.

Order Number	Reference Part Number	Inner Diameter μm	Description
Denuding Pipettes			
5 Vials of 10 (50 Total)			
G46021	K-FPIP-1120-10BS-5	120	oocyte/embryo manipulation
G26711	K-FPIP-1130-10BS-5	130	oocyte/embryo manipulation
G26712	K-FPIP-1140-10BS-5	140	oocyte/embryo manipulation
G26713	K-FPIP-1170-10BS-5	170	oocyte/embryo manipulation
10 Vials of 1 (10 Total)			
G46019	K-FPIP-1140-10	140	oocyte/embryo manipulation
G46018	K-FPIP-1170-10	170	oocyte/embryo manipulation
Manipulation Pipettes			
1 Vial of 10 (10 Total)			
G26181	K-FPIP-1080-10BS	80	blastomere or polar body manipulation
G26718	K-FPIP-1100-10BS	100	oocyte/embryo manipulation
G18878	K-FPIP-1300-10BS	300	blastocyst handling
5 Vials of 10 (50 Total)			
G46020	K-FPIP-1300-10BS-5	300	blastocyst handling
G26057	K-FPIP-1600-10BS-5	600	manipulation of oocyte-cumulus complex
10 Vials of 1 (10 Total)			
G46017	K-FPIP-1300-10	300	blastocyst handling

COOK **Microcise™ Fleximate™**
MEDICAL FLEXIPET® ASPIRATION TOOL

Order Number	Reference Part Number	Description
10 Pouches of 5 (50 Total)		
G55460	K-FMDT-1000	provides vacuum for Flexipet; 100% silicone

Intended for one-time use.
Not all part numbers shown on this product information sheet may be approved for sale in all regulatory jurisdictions. Consult with your local Cook representative or customer service center for details.

Customer Service Centers

- AORTIC INTERVENTION
- CRITICAL CARE
- ENDOSCOPY
- INTERVENTIONAL RADIOLOGY
- LEAD MANAGEMENT
- PERIPHERAL INTERVENTION
- SURGERY
- UROLOGY
- WOMEN'S HEALTH

EMEA: EDI - www.cookmedical.com/edi.do
 Distributors: +353 61239240, ssc.distributors@cookmedical.com
 Austria: +43 179567121, oe.orders@cookmedical.com
 Belgium: +32 27001633, be.orders@cookmedical.com
 Denmark: +45 38487607, da.orders@cookmedical.com
 France: +33 171230269, fr.orders@cookmedical.com
 Germany: +49 6950072804, de.orders@cookmedical.com
 Hungary: +36 17779199, hu.orders@cookmedical.com
 Ireland: +353 61239252, ie.orders@cookmedical.com
 Italy: +39 0269682853, it.orders@cookmedical.com
 Netherlands: +31 202013367, nl.orders@cookmedical.com
 Norway: +47 23162968, no.orders@cookmedical.com
 Poland: +48 223060159, pl.orders@cookmedical.com
 Spain: +34 912702691, es.orders@cookmedical.com
 Sweden: +46 858769468, se.orders@cookmedical.com
 Switzerland - French: +41 448009609, fr.orders@cookmedical.com
 Switzerland - Italian: +41 448009609, it.orders@cookmedical.com
 Switzerland - German: +41 448009609, de.orders@cookmedical.com
 United Kingdom: +44 2073654183, uk.orders@cookmedical.com
www.cookmedical.com

Americas: EDI - www.cookmedical.com/edi.do
 Phone: +1 812.339.2235, 800.457.4500, Fax: 800.554.8335,
 E-mail: orders@cookmedical.com
Australia: ABN: 51 141259118
 Phone: +61 738411188, 1800777222, Fax: +61 738411288, 1800077283,
 E-mail: wacustserv@cookaust.com.au

